Inkontinenztherapie-Stimulator  
Stimulator for Incontinence-Therapy

# BioBravo

Medizinprodukt / Medical Device

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# Content

Please read the Operators Manual carefully before using the device  
and pay attention to all references !

	<b>Page</b>
<b>I. Incontinence Therapy BioBravo</b>	
Delivery & Accessories / Medical Device Reporting	31
Introduction: What is <b>BioBravo</b>	32
Indications	32
Contraindications	33
Warnings	34
Advice on Application of Electrodes	35
Technical Data	36
Safety Adjustments	36
Explanation of Symbols	37
Maintenance	37
Warranty	38
Real Time Clock Setup	38
<b>II. Elements of the BioBravo</b>	
Display and Operating Elements	39
Description of the Device	40
Battery Compartment with LOCK-Button	41
<b>III. Application of the BioBravo</b>	
Preset Programs P01 - P09	42-43
Setting the Custom Programs	44-45
<b>IV. Incontinence-Therapy</b>	
What is Incontinence	46
Different Types of Incontinence	47
Programs 01-09	48-52
<b>V. Control of Therapy</b>	
Lock-/Unlock-Switch	53
Statistics	53
Reading the Statistics	54-55
Trouble Shooting Guide	56
<b>Waste Disposal - Recycling of Batteries - Literature - Copyright</b>	57
<b>Anatomy</b>	58
<b>Electromagnetic Compatibility</b>	62-64

## Delivery & Accessories

### Included in the delivery:

REF 010E-003: 1 device **BioBravo**, 2 lead wires, 4 self adhesive electrodes (40 x 40 mm), 4 self adhesive electrodes (40 x 80 mm), 4 batteries type AA and 1 Operators Manual

With regular average use, the expected service life of **BioBravo** is 10 years. Service life of accessories is one year.

### Accessories:

REF

011E-006: 1 set self adhesive electrodes 40x40 (4 pieces)

011E-007: 1 set self adhesive electrodes 40x80 (4 pieces)

011E-051: 1 set self adhesive electrodes 32mm Ø (4 pieces)

011E-008: 1 vaginal probe “IncoProbe V”

011E-009: 1 rectal probe “IncoProbe R”

### Spare parts:

REF

012E-013: 2 lead wires

052F-805: 4 batteries type AA

012F-037: 1 tube electrode jelly for probes

012E-047: 1 operators manual **BioBravo**

012E-086: 1 carrying case

**BioBravo** shall only be used with original accessories.

The batteries life is approximately 12 hours or more at 20 mA per channel. The storage time of the batteries is approximately 2 years. The service life (expiry date) of electrodes is printed on each package.

Max. environmental conditions for transportation: -25 to +70 degree centigrade, 0-93% atmospheric humidity. Recommended environmental conditions for storage: -25 to +40 degree centigrade, 0-93% atmospheric humidity. The device shall not to be stored in direct sunlight.

Allow up to one hour for the device to return to normal operating temperature if stored in conditions below +5°C or above +40°C.

Environmental conditions for use: +5 to +40 degree centigrade, 0-93% atmospheric humidity, air pressure 700-1060 hPa. During use the surface temperature of the device can be 41°C or higher, but never exceeds 48°C.

### Medical Device Reporting:

If there is a serious deterioration in the state of your health when using the medical device, report this to your distributor or to the manufacturer and to the competent authorities of your country.

## Introduction – What is BioBravo ?

**BioBravo** is a neuromuscular stimulator, which has been developed according to latest findings by MTR+ in consultation with medical professionals and patients. Doctors and patients are the intended operators for this unit. This device is a modern 2-channel stimulator, which is often used with vaginal- or rectal-probes. Additionally this unit allows the use of 2 pairs of skin-electrodes (= 4 electrodes) to work simultaneously, as well.

**BioBravo** is a stimulation device, which allows both, high-efficient muscle stimulation for the pelvic muscles as well as being suitable for the calming of the hyper nervous bladder. This device can therefore be used for all forms of urinary and/or faecal incontinence without any assistance. Another program is designed for the treatment of intestinal stasis. It can be used if there is no mechanical obstruction.

Additionally the three configurable programs enable the specialist to compile specially tailored programs for their own individual purpose. This characteristic of the **BioBravo** offers each physician or physiotherapist, and also the experienced patient, the possibility of a focused coordinated treatment for the respective state of health. These programs, after storing, can be changed at any time, for example to be altered according to the progress of therapy, an almost unrestricted variation range is available with this device to the user.

The clip on the back of the unit allows to fasten the device at the clothes, but the unit is intended to be supported by hand during normal use. The belt clip is for temporary relief only.

## Indications

**BioBravo** was designed for the following indications:

- Elimination of light to mid strength faecal incontinence (PRG 7, 8)
- Elimination of light to mid strength urinary incontinence
  - Urge-incontinence (PRG 1, 2)
  - Stress-incontinence (PRG 3, 4)
  - Mixed-incontinence (PRG 5, 6)
- Increase of intestinal activities (PRG 9)

## Contraindications

### **BioBravo may not be used in the following cases:**

- By patients with electronic implants like pumps or a pace maker as functional disturbances can occur.
- By patients with myocardial damage (damage to the cardiac muscle) or arrhythmia (inconsistent heart beat).
- In presence of flammable anesthetic mixtures with air or with oxygen or nitrous oxide.
- When operating machinery with a potential of danger.
- While driving a vehicle.
- By patients with epilepsy.
- If severe local inflammation or thrombosis is present.
- By patients with mechanical ileus.
- By patients with febrile illnesses or infectious diseases.
- Do not use with a surgical high frequency apparatus because burn marks could occur underneath the electrodes.
- When using metal-containing contraceptive spirals.

### **Please pay unconditional attention to the following basic rules:**

- Do not stimulate transcerebral (via the head).
- Do not stimulate via the Carotis-Sinus-Nerve (carotid artery).
- Do not stimulate in the area of the Sinus knot (heart).
- Do not stimulate on the Bulbous (eye).
- Do not stimulate in the Larynx-Pharynx-Area (throat / mouth).
- Do not stimulate in places with skin defects.
- Do not position the electrodes directly on the spine.

### **Take special care with the use of the electric stimulation:**

- if after an operation the healing process can be interrupted through a muscle contraction.
- after acute bleeding or bone fractures with a tendency to bleed.
- during menstruation or pregnancy.
- in case of desensitized skin with nerve damage.
- if stimulation is located close to osteosynthesis material (metal in the body).
- if patients cannot use the device properly, due to mental or physical limitations.

## Warnings

**BioBravo** is type BF equipment and may not be immersed in liquids. The **BioBravo** is protected against debris larger than 12.5 mm in diameter from entering the cabinet, is not water resistant and rated IP20.

Burn marks could occur underneath the electrodes if used with surgical high frequency apparatus.

If device operation occurs close to short wave, microwave, cellular or other wireless equipment, instability of the values of the stimulator could occur causing a temporary reduction or loss in current. Move away from the source of interference and increase the current to continue the program.

Strong static discharge occurring to the device may interfere with its operation. Turn off the device and turn on again after a couple of seconds if device effected by a static discharge.

The application of the electrodes nearby the thorax can increase the risk of cardiac fibrillation.

If in the case of treatment with the probe, pain or irritation appear in the area of the pelvis, or bleeding in-between menstruation, interrupt the therapy and consult your doctor.

Patients with metal and/or electronic implants may only stimulate with permission of an experienced doctor.

Stimulation in the areas of tumors is allowed only by instruction of the specialized physician.

The **BioBravo** may not be operated if the battery compartment is not closed.

Before opening the battery compartment, make sure that there are no lead wires in the jacks at the head of the unit to eliminate any possibility of current flow.

The operator should not touch the battery contacts & patient simultaneously.

Never connect the device directly to a battery charger or to any other mains powered equipment.

If using rechargeable batteries be sure to use a CE approved battery charger.

If you have swallowed any part of the device or any of its accessories, please contact a doctor immediately.

TENS and muscle stimulation should be used during menstruation or by pregnant women as well as by children under the guidance of a doctor or therapist only.

Please don't use a probe during menstrual period or pregnancy.

Remove batteries and battery cover, drain water from device and allow (BioBravo) to dry normally for 48 hours if accidentally submerged in water.

**Please keep this device stored out of the reach of children. Do not leave the lead wires in reach of children, since strangulation is possible due to the length of the lead wires.**

## Advice on Application of Electrodes

Please empty the urinary bladder and clean the area to be treated.

Make sure that **BioBravo** is turned off (no indication on the LCD display).

Now connect the single poled ends of the lead wires with the vaginal or rectal probe or the skin-electrodes.

Then fix the output plug(s) of the lead wires into channel A or B of **BioBravo**.

Furthermore make sure that all plugs fit completely into the electrode sockets as well as the device sockets.

If you are using a probe, lubricate it with water or a gel suitable for electrotherapy.

Please don't use any oil, ointment or cream, since all these have an isolating effect.

Now place the probe or the electrodes according to the instructions of your doctor or physio-therapist.

In the case of the skin gel losing its adhesion, you can't achieve more than 6 mA - please replace the electrodes. The same effect, however, also occurs when using a damaged lead wire.

Therefore, if the normal treatment is not possible by using new electrodes, the lead wire may be faulty.

If using a probe, please clean it carefully before and after use with a mild detergent (without alcohol) and ensure the probe is rinsed thoroughly to remove all cleaning residues. Ensure the connectors are not submerged in water as they may corrode. Ensure the probe is completely dry before storage.

Electrodes/probes which were damaged due to an improper handling may not be used.

Electrodes/probes with a damaged lead wire isolation (e.g. visible copper wire) may not be used.

Electrodes/probes may not be altered (e.g. by means of scissors or other tools).

Electrodes with damaged, missing or stained aluminium coating or carbon layer, also with insufficient adhesive gel may not be used.

Do not pull the electrodes/probes by force on the electrode lead wire.

Apply the electrodes on clean and health skin only.

In the case of strong bodily hair growth, the hair should be shortened with scissors.

Neither stretch the skin nor the electrodes when applying the electrodes.

To remove the electrodes from the skin, please lift the electrodes at the rim towards the middle.

Effective current densities of more than 2 mA/cm<sup>2</sup> require an increased attention of the user with regard to the intensity adjustment.

We recommend the use of our vaginal- or rectal-probes.

If you use self-adhesive electrodes, we recommend the use of our self-adhesive electrodes with an area of 40x40 or 40x80 mm.

All electrodes or probes are made for use of one single person only.

Electrodes can contain amounts of nickel. Allergy sufferers should check this before using.

Should skin irritation occur whilst using the **BioBravo**, please interrupt the application and inform your doctor - to exclude an allergic reaction.

## Technical Data

Stimulator with 12 programs

Channels: 2 isolated independent channels

Waveform: Asymmetrical, rectangular, biphasic with zero DC-current

Intensity: 0 - 90 mA into 500 ohm load,

70 mA maximum into 1000 ohm load

65 mA maximum into 1500 ohm load

(Indication only, actual intensity varies depending on condition of the electrodes)

Classification: Internally powered, constant current, continuous operation, type BF applied part, class IIa, IP 20

Frequency: 1 - 200 Hz

Pulse Width: 50  $\mu$ s - 450  $\mu$ s

Ramp up/down: 0,1 - 9,9 seconds

Time adjustment: 1 minute - 9 hours, 59 minutes

Power supply: 4 batteries 1,5 V type AA

Voltage max.: 180 V without load (80 V into 1000 ohm load)

Dimensions: L 141,6 mm x W 74 mm x H 40,4 mm

Weight: 240 g (incl. batteries)

Firmware version 3.5 or later

This device is quality controlled and fulfills all legal requirements for medical products.

Therefore this device carries the CE-sign „CE 0123“. The serial number is on the rear of the unit.

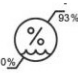
**Manufacturer:** MTR+ Vertriebs GmbH,  
Kamenzer Damm 78, 12249 Berlin, Germany  
[www.mtrplus.com](http://www.mtrplus.com)

## Safety Adjustments

1. The +buttons become inactive after one minute running time of a program to avoid an unintentional increase of intensity . This lock is switched off again by pressing the +button three-times repeatedly. Now an increasing of intensity is possible again for another minute.
2. If the electrical circuit should be interrupted during operation ( $> 6$  mA), for example by losing an electrode or by a broken lead wire, the device stops stimulating immediately.
3. If any phase comes to its end the intensity falls down to 8 mA automatically and must be set by the user to the desired setting.

## Explanation of Symbols

	Attention Warning		Manufacturer
	Typ BF applied parts		Date of manufacture 2022 - 04
	Serial-Number of the product		CE-mark: conforms to essential requirements of the European standards
	Keep dry		Not suited for persons with cardiac pacemaker
	Batch Code		Read the instructions
	Medical Device		Unique Device Identification
	Temperature limitation (transport)		Keep away from sunlight
	Temperature limitation (storage)		Humidity limitation

## Maintenance

**BioBravo** has been designed to be maintenance free. Nevertheless, please adhere to the following advice:

If needed, please clean **BioBravo** with a mild detergent, water and a damp cloth. Do not expose **BioBravo** to extreme humidity and do not submerge under water.

You may clean the lead wires with a damp cloth. You may also from time to time treat the lead wires with talcum powder in order to prevent cracks and to improve the lead wires life.

A clean cloth, moistened with an alcohol free disinfection liquid can be used to clean the device. Please follow all instructions of the manufacturer of the liquid.

After termination of the therapy, **BioBravo** can be re-used by other patients. Please clean the device with a mild disinfectant cleaner (s.a.).

The **BioBravo** does not require calibration. The **BioBravo** is not repairable and contains no user serviceable parts. No modification to the **BioBravo** is allowed.

When changing the batteries, please make sure to insert the new batteries within 30 minutes after removal of the spent batteries. We recommend that you always have spare batteries with the device.

## Warranty

MTR+ Vertriebs GmbH provides a 24 month warranty, commencing from the date of our delivery (invoice date), that in case of failure, the **BioBravo** will be serviced or replaced free of charge. All returns must first be authorised by MTR+ Vertriebs GmbH in advance.

Excluded from this warranty are failures of the **BioBravo** which result from excessive use, careless or improper handling as well as damage by force.

The warranty loses its lawful meaning if the producer MTR+ Vertriebs GmbH has not given its consent in writing prior to repairworks or servicing being carried out. Without this written consent the manufacturer will not return the device.

The manufacturer reserves the right of technical changes, which may serve to increase treatment success, the functionality of the device or its accessories.

## Real Time Clock Setup

**BioBravo** has a “Real Time Clock”, that shows year, month, day, hours and minutes to store detailed statistics of treatment data.

To set time and date, please select any custom program 10-12 and press the **SET**-button for at least 5 seconds. You’ll reach the configuration-mode. Now press **SET** once again for 10 seconds.

With the +/- button of channel A (left side) you can step forward to the following values

- hour (indicated by: H)
- minute (indicated by: MI)
- day (indicated by: dA)
- month (indicated by: MO)
- year (indicated by: Y)

The respective value is flashing and can be changed with the +/- buttons of channel B (right side). Finally you save all your settings by pressing the PRG-button once again.

Before starting the therapy, please set the clock once if necessary.

If **BioBravo** has been without batteries for more than 30 minutes, the clock must be reset.

## Display and Operating Elements

**„ON/OFF“:** By pressing the button once, the device will either be switched on or off.

**„+/- buttons“:** These buttons adjust the intensity for channels A and B. The device has two independently controllable output channels A and B. When pressing the respective arrow buttons the intensity will either be increased or decreased in steps of 1 mA. 60 seconds after the last intensity regulation, a further increase may not be carried out due to an electronic barrier. Pressing the „+“ button three times, will release the lock and the intensity can be increased again. A reduction of the intensity is possible at any time.

Whilst configuring the programs 10 – 12 one switches between the programmable parameters by pressing the left „+/-“ button (channel A), and through pressing the right „+/-“ button (channel B) the desired value/data for respective parameter can be set. In this way, the application time, the stimulation, the pause, rising pulse and falling pulse, as well as synchronous or asynchronous stimulation can be selected.

**„PRG“:** With this button the preset programs 01–09 or the three customizable programs 10–12 can be selected for individual use.

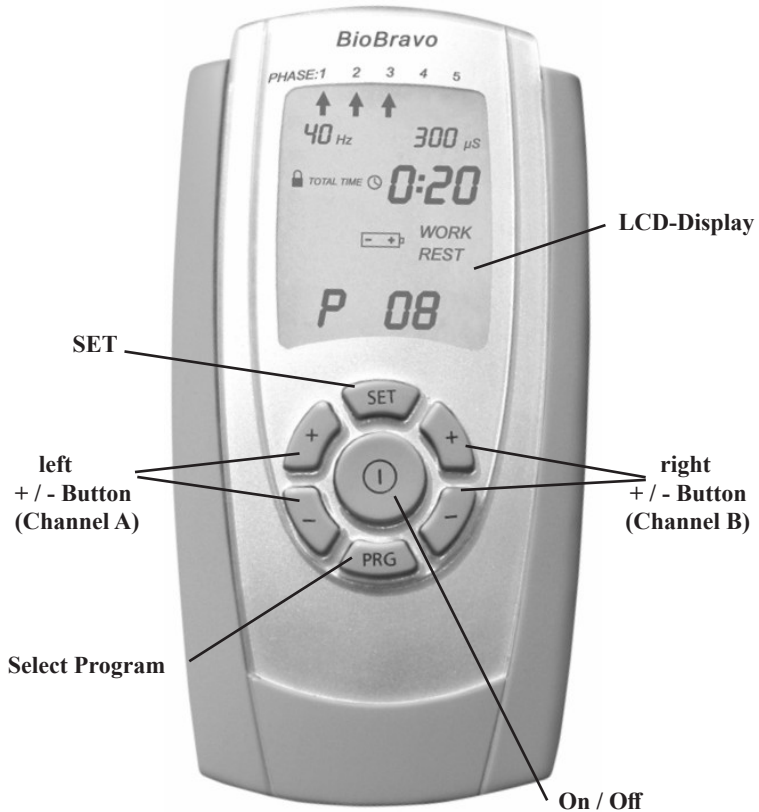
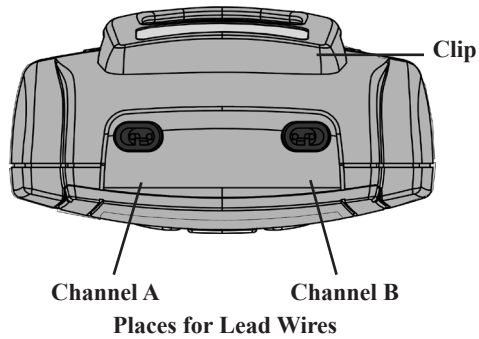
Pressing the PRG-button once at any time during a running program will reduce the mA on both channels to zero and pause the clock display with the actual treatment time. The clock continues to show the time of further treatment but it is frozen with the “:” symbol no longer flashing. In addition, this button stores the current adjustment in the programming mode and ends it. Then the device returns to the home screen of the selected program.

**„SET“:** With the SET-button you can select the different program phases. With each press of this button, the phase in progress is terminated and the next program phase is started. This applies to either the running of the programs 04-06 and 08-09 and also to the programming and the sequence of programs 10–12. Programs with one single phase will be paused by pressing the SET-button.

If you keep the SET-button pressed for 5 seconds whilst in programs 10–12, the symbol of the clock begins flashing and you will be able to start programming the individual settings. Pressing SET once again for 10 seconds allows to setup the RTC.

Holding the „SET“ button for 5 seconds while the statistics are displayed, will display „dEL“. Holding „SET“ a second time for 5 seconds will delete the statistics, set all custom programs to default values and show the RTC setup screen.


## Description of the Device

## Battery Compartment with LOCK-button

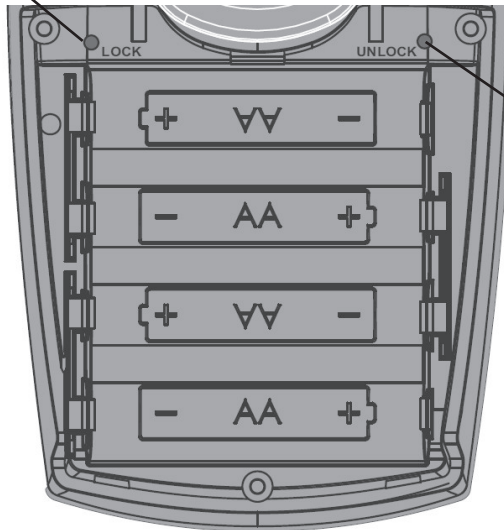**BioBravo** allows the doctor or therapist to lock one program for the patient to use (s. p. 51). In this way the medical practitioner can be quite sure that the patient only uses the device with the correct program. The LOCK-button can be pressed with a pin of the lead wire or any other suited thin object. Pressing will fix the selected program so that different programs cannot be used anymore.

UNLOCK button removes the LOCK of programs and shows global statistics

Arrangement of the batteries

## Preset Programs 01–09

### Preparations:

Make sure that **BioBravo** is switched off. Cleanse the area of the skin which is to be treated with clean water. Following that, make sure that all plugs are completely connected to the respective electrode sockets as well as device sockets. Place the vaginal-/rectal-probe or the adhesive electrodes onto the area that has been prior cleansed, according to the advices in pages 33-35 of this manual.

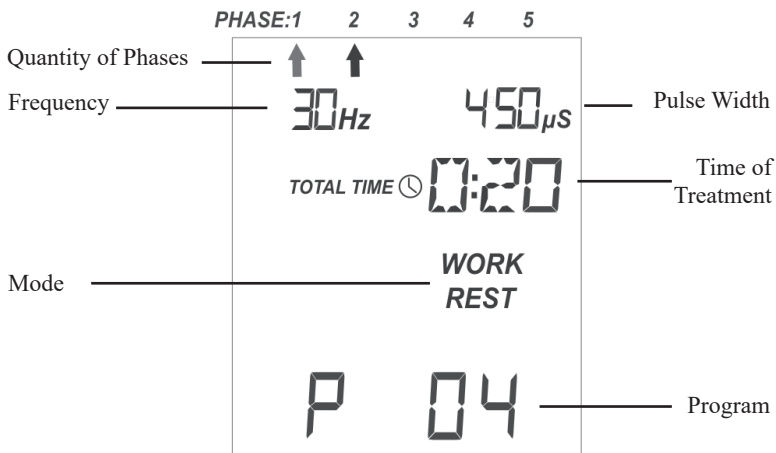
**BioBravo** is operated by 4 batteries of type AA. Open the battery compartment at the back by pushing down the lid and insert the batteries. Please ensure the correct polarity in accordance with the markings. **BioBravo** is now operational. If the battery voltage is too low, the battery-symbol in the LCD-display will flash. Please replace the battery at this time. Empty batteries constitute special waste and therefore have to be disposed of at an appropriate collection point. Never dispose of batteries via domestic or household waste.

### Adjustment:

1. Press the button „ON/OFF” once.


**BioBravo** is now switched on. The last used program appears on the LCD-display.

Parameters programmed for the first phase will be displayed as well as the quantity of all phases, the actual treatment mode and the total treatment time.

## Preset Programs 01–09

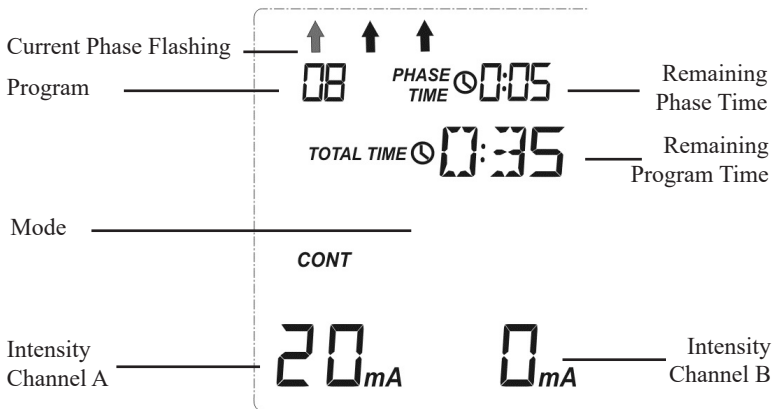- By pressing the program button „PRG” you can choose from the programs 01 – 09 (pre-programmed).
- This is followed by pressing the buttons „+/-” below the channel used.  
Now the selected program runs automatically and the LCD-screen changes.

The current phase will be indicated by an flashing arrow and in the upper section of the LCD you can see the remaining phase time. In the bottom line the intensity used in each channel appears. If the button „+” is pressed repeatedly the intensity of stimulation increases. If the button „-” is pressed repeatedly the intensity of stimulation is reduced. For the setting of the intensity it is vital that one feels the electric current but that discomfort is never felt. Please take into account that for safety reasons the intensity falls back to 8 mA at the beginning of each new phase and must be increased again.

Each phase end is indicated by 5 peep tones.

- If you wish to interrupt the program, then press the button „PRG” once. The program will run again for the remaining program time when pressing the button „+”. In this case, the intensity must be reset to the desired level.
- By pressing the „SET” button you end the current phase and move to the next. Programs with one single phase will be finished.
- BioBravo** is switched off when pressing the button „On/Off” or automatically four minutes after the program is finished.

## Setting the Custom Programs 10-12

1. Switch on **BioBravo** by pressing the „ON/OFF” button.
2. Choose the desired program (10-12) with the „PRG” button.
3. Press „SET” button for at least five seconds. **BioBravo** changes to the programming mode and the upper watch, which indicates phase duration, flashes to show that this function can be altered.

**In the following please use the left „+” button to store your choice and to change to the next step. The right „+/-” buttons are respectively used in order to alter the chosen and flashing parameter.**

4. Select with the right „+/-”buttons the desired time for the first phase.
5. By pressing the left „+”button you will be able to store your choice and change to the next step. In the display the mode now flashes, which you adjust according to your wishes by pushing the right „+/-”buttons.

If you have chosen „CONT” or „BURST” you can determine frequency and after storage and change with the left +button respectively the pulse width by pushing the right „+/-”buttons.

If you have chosen „WORK/REST”- rhythm you can determine the following parameters one after the other by pushing the right „+/-”buttons.

- frequency (**Hz**)
- pulse width (**µS**)
- work time (**sec**)
- rest time (**sec** = recovery time)
- increase / decrease ramp up time of the pulse (**RU**)
- increase / decrease ramp down time of the pulse (**Rd**)
- synchronous (**SY**) or alternating (**AL**) output of the channels A and B. If you have selected **“SY”** you can program **delay-time (dL)** of 0 sec. up to 5 sec. from channel A to B.

If you have chosen „MOD” you can alter the following parameters one after the other.

- low limit of frequency (**Hz LO**)
- upper limit of frequency (**Hz HI**)
- low limit of pulse width (**µS LO**)
- upper limit of pulse width (**µS HI**)
- low limit of intensity (AdJ) - the display in the right corner below shows values from 0.5 to 1.0. This adjusts the intensity of current within the modulation time between 50% (=0.5) and 100% (=1.0) of the original set mA-value. The intensity, of a modulated program, can never increase above the 100% of the set value.
- modulation time (possible choice: 2 sec. up to 60 sec)

## Setting the Custom Programs 10-12

6. Push the „SET” button to select the second phase of the program. You can program up to 5 phases of your own in each of the three configurable programs. If you want to reduce a configured program of 5 phases to less phases, you simply set the time of the first unwanted phase to zero.
7. The programming is completed by pressing the „PRG” button and the program is stored.

### Selectable frequency and pulse width settings:

Continuous-Mode: 1 to 200 Hz, 50 to 450  $\mu$ S

Burst-Mode: 35 to 200 Hz, 50 to 200  $\mu$ S

Modulated-Mode: 1 to 200 Hz, 50 to 450  $\mu$ S

Work/Rest-Mode: 1 to 200 Hz, 50 to 450  $\mu$ S

**It is not possible to program more than 300  $\mu$ S if the frequency is above 100 Hz.**

### The settings can be made as follows.

- Frequency: 1, 2, 3, 4, 5, 6, 7, 8, 9, 10, 12, 14, 15 Hz, and then increasing in 5 Hz steps until max. 200 Hz
- Pulse width; 50 to 450  $\mu$ S in steps of 10 microseconds
- Intensity: 50-100% in steps of 10%
- Modulation time: 2, 3, 4, 5, 6, 7, 8, 9, 10, 12, 15, 20, 30, 40, 50, 60 seconds
- Work time: 2-99 seconds in steps of 1 second
- Ramp up time: 0,1-9,9 seconds in steps of 1 second
- Ramp down time: 0,1-9,9 seconds in steps of 1 second
- Rest time: 2-99 seconds in steps of 1 second
- Delay: 0,0-5,0 seconds in steps of 0,1 second
- Session time: 1-599 minutes in steps of 1 minute

## Incontinence Therapy

What is Incontinence ?

The expression “incontinence” refers to the inability to voluntarily withhold urine and/or stool. The causes of incontinence are malformations, illness or injury.

Urinary incontinence is more common than one might initially suspect. It looks that the number of people affected by incontinence will increase steadily. By the year 2050 according to various estimates nearly 30 % of people could be affected by incontinence. About 80% of women between 20 and 75 years and 10% of men suffer from urinary incontinence. Faecal incontinence in both sexes is about 10%.

Incontinence is not only a medical problem, but also represents a significant psychological burden. Many affected individuals respond with depression and withdrawal from social life. Incontinence should never be tolerated as fatefully, since most patients can be helped with proper treatment. Forms of incontinence, upon which medicines have little effect are therapeutible by electric stimulation.

Therapy by electric stimulation is not stressful, it is simple in its application and allows wide ranging use for treating bladder and discharge malfunction as well as faecal incontinence.

The flow of electricity that is generated by means of probes or adhesive electrodes, leads to a pelvic floor contraction, which the patients feel. They learn quickly to control the muscles of the pelvic floor themselves. The improvements can be measured by the appropriate use of a biofeedback device. Experience has shown that you can achieve significant therapy progress within a short time quite often, by using electric stimulation combined with biofeedback.

The aim of the electric stimulation is as follows:

- focused training of the pelvic muscles
- strengthening of the bladder closing muscle
- attenuates the highly active bladder muscle
- regaining the physiological sensation of the bladder
- strengthening of the sphincter muscle
- increase of intestinal activities

## Different Types of Incontinence

**BioBravo** can be applied with the following types of incontinence.

Stress incontinence means that the muscles that belongs to the closing mechanism of the bladder exit are not fully functionable. When pressure rises (e.g. through sneezing, coughing, laughing or physical exertion) the loss of urine will occur even without micturition. The cause is a weakening of the pelvic muscles which can result from either weakness of the connective tissue, birth or prostate surgery.

Stress incontinence is classified according to three severity levels:

- **Grade 1:** Involuntary loss of urine during violent increase of pressure in the abdominal area, e.g. by sneezing, coughing, laughing or heavy physical exercise.
- **Grade 2:** Involuntary loss of urine in moderate increase of pressure in the abdominal area, e.g. by walking or climbing stairs.
- **Grade 3:** Involuntary loss of urine with only a slight increase in pressure in the abdominal area even when lying.

Urge incontinence means that the closing mechanism of the bladder and urethra are mostly functioning. Often it comes to extrude movements of the bladder, which cannot be suppressed through will power. One speaks of a hyper active bladder muscle. Micturition is uncontrollable and the pressure of the bladder increases to the point, that the resistance of the healthy muscular closing system will be overcome. Possible causes are disorder of the superior nervous system.

A mixed form of stress and urge incontinence means that the cause lies in the weakness of the muscular closing system as well as a hyperactive bladder muscle. In most cases one of the two forms is especially strong.

Faecal incontinence means a disorder of the function of the sphinkter anus externus. A controlled discharge of bowel contents is not possible anymore. The causes for the functioning disorder of the sphinkter anus externus are manifold. There are various different levels:

- **Grade 1:** Uncontrolled disposal of winds / Light soiling of the laundry
- **Grade 2:** Uncontrolled discharge of fluid stool / Uncontrolled disposal of winds / Occasional uncontrolled stool
- **Grade 3:** Loss of stool and all the winds totally uncontrolled

## Programs

Progr.: 01		Urge incontinence 1				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	20				
Mode		Cont				
Frequency work	Hz	10				
Frequency rest	Hz					
Pulse duration	$\mu$ S	250				
Modulation time	secs					
Ramp up time	secs					
Ramp down time	secs					
Work time	secs					
Rest time	secs					
Overall time	20 min					

Calming of the hyperactive bladder.

Progr.: 02		Urge incontinence 2				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	20				
Mode		W/R				
Frequency work	Hz	10				
Frequency rest	Hz					
Pulse duration	$\mu$ S	350				
Modulation time	secs					
Ramp up time	secs	1				
Ramp down time	secs	1				
Work time	secs	6				
Rest time	secs	15				
Overall time	20 min					

Calming of the hyperactive bladder.

## Programs

Progr.: 03		Stress incontinence 1				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	20				
Mode		W/R				
Frequency work	Hz	30				
Frequency rest	Hz					
Pulse duration	µS	200				
Modulation time	secs					
Ramp up time	secs	0,8				
Ramp down time	secs	0,8				
Work time	secs	5				
Rest time	secs	8				
Overall time	20 min					

Training the weakened pelvic muscle. This program can be used with faecal-incontinence as well.

Progr.: 04		Stress incontinence 2				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	10	10			
Mode		W/R	W/R			
Frequency work	Hz	30	40			
Frequency rest	Hz					
Pulse duration	µS	450	300			
Modulation time	secs					
Ramp up time	secs	1	1			
Ramp down time	secs	1	1			
Work time	secs	6	6			
Rest time	secs	15	15			
Overall time	20 min					

Training the weakened pelvic muscle. This program can be used with faecal-incontinence as well.

## Programs

Progr.: 05		Mixed incontinence 1				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	3	10	5	4	3
Mode		W/R	W/R	W/R	W/R	W/R
Frequency work	Hz	3	10	20	30	40
Frequency rest	Hz					
Pulse duration	μS	250	250	250	200	200
Modulation time	secs					
Ramp up time	secs	0,8	0,8	0,8	0,7	0,7
Ramp down time	secs	0,8	0,8	0,8	0,7	0,7
Work time	secs	4	4	4	4	4
Rest time	secs	4	4	4	6	6
Overall time	25 min					

Treatment of mixed urethral incontinence. Both the overactive bladder and the weakened pelvic muscle are treated.

Progr.: 06		Mixed incontinence 2				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	10	10	10		
Mode		W/R	Cont	W/R		
Frequency work	Hz	12	10	35		
Frequency rest	Hz					
Pulse duration	μS	400	200	300		
Modulation time	secs					
Ramp up time	secs	1		1,5		
Ramp down time	secs	1		1		
Work time	secs	6		5		
Rest time	secs	15		12		
Overall time	30 min					

Treatment of mixed urethral incontinence. Both the overactive bladder and the weakened pelvic muscle are treated.

## Programs

Progr.: 07		Faecal incontinence 1				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	20				
Mode		W/R				
Frequency work	Hz	40				
Frequency rest	Hz					
Pulse duration	$\mu$ S	200				
Modulation time	secs					
Ramp up time	secs	1				
Ramp down time	secs	1				
Work time	secs	6				
Rest time	secs	15				
Overall time	20 min					

Training the sphincter muscle. This program can be used with stress-incontinence as well.

Progr.: 08		Faecal incontinence 2				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	5	10	5		
Mode		W/R	W/R	W/R		
Frequency work	Hz	40	35	30		
Frequency rest	Hz					
Pulse duration	$\mu$ S	300	400	450		
Modulation time	secs					
Ramp up time	secs	1	1	1		
Ramp down time	secs	1	1	1		
Work time	secs	5	6	5		
Rest time	secs	12	12	12		
Overall time	20 min					

Training the sphincter muscle. This program can be used with stress-incontinence as well.

## Programs

Progr.: 09		Treatment of intestinal stasis				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	10	10			
Mode		W/R	W/R			
Frequency work	Hz	8	12			
Frequency rest	Hz					
Pulse duration	μS	450	400			
Modulation time	secs					
Ramp up time	secs	1	1			
Ramp down time	secs	1	1			
Work time	secs	5	6			
Rest time	secs	10	12			
Overall time	20					

In this program 09 it is tried to increase intestinal activity, if there is no mechanical ileus. To reach this target one uses both channels of the **BioBravo** in connection with self-adhesive electrodes, which are fixed crosswise on the body in the intestines area.

### Abbreviations:

AdJ:	Adjustment of intensity
AL:	Alternate function of the channels A and B
BURST:	The burst mode delivers 2 blocks of 9 impulses each per second
CONT:	In the continuous mode a permanent unchanged current is used
dA:	Day
dEL	Delete
dL:	Delay between channels A and B
HAN:	Change of frequency and pulse width in a 3:3 seconds rhythm (indicated as CONT + BURST)
HI:	Upper value (high)
Hz:	Hertz is the measurement unit for the frequency of impulses per second.
LO:	Lower value
mA:	Milli-amp is the measurement unit for the electrical current intensity.
MOD:	In the modulated mode the frequency and/or the pulse width and/or the intensity changes in a predefined rhythm automatically.
Rd:	Ramp down
RU:	Ramp up
SEC:	Seconds
SY:	Synchronous function of the channels A and B
TI:	Time
TM:	Time of modulation-interval
W/R:	Work/Rest mode
μS:	Microsecond is the measurement unit for the length of every single impulse.

## LOCK-/UNLOCK Buttons

The **BioBravo** offers the ability to lock one program in place so that other programs or settings cannot be selected or changed. This allows the doctor or therapist to ensure that the patient uses only the program that has been selected for him.

The LOCK switch in the battery compartment (s. p. 41) can be used by means of the pin of the electrode lead wire. To do this, please disconnect all lead wires from the device, switch on the device and open the battery compartment. After pressing LOCK you hear a single beep and the selected program is locked. In this case, you cannot use any other program anymore until UNLOCK is pressed..

Pressing the UNLOCK button in the battery compartment (s. p. 41) releases the program lock. A second press shows the global statistics on the display. If the unit was not locked, these statistics are already displayed on the first press.

## Statistics

The therapist can read information about the compliance of the patient on the display at any time he wants. This way it is possible to evaluate whether the device was used according to the instructions of the doctor (quantity, intensity, etc.)

**BioBravo** offers the chance to get a quick survey about the stored data of all treatment-sessions.

The global statistics display the following averages:

- number of treatments per day
- number of days in use
- most used program
- total time of use
- intensity per channel

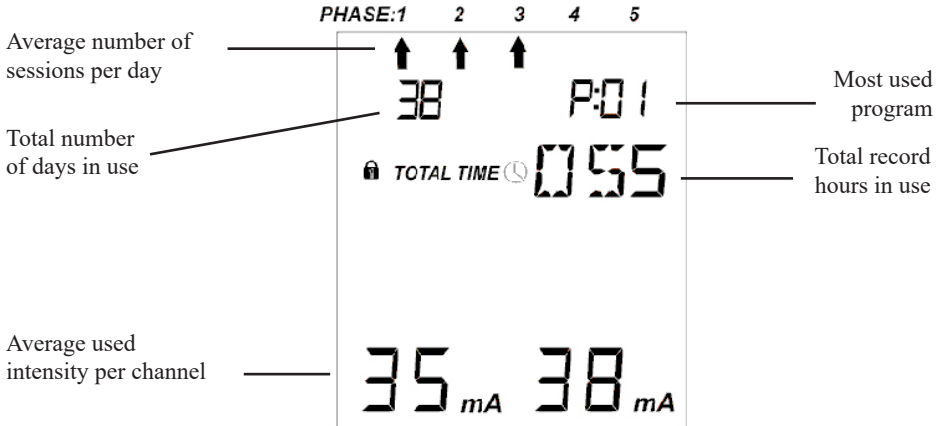
If more detailed information is required, **BioBravo** allows the therapist to have a closer look at each single session of the last 60 treatment-days. You can see:

- number and date of the treatment-day
- number of session at this day
- time of session-start
- number of program used in this session
- total-time of the session
- intensity used for each channel in this session


## Reading the Statistics

If you want to read the memory, switch on the device and press the UNLOCK-button in the battery compartment twice if the unit was locked in one program or once if it was not locked. The global statistics can be viewed on screen now.

### Global Statistics:

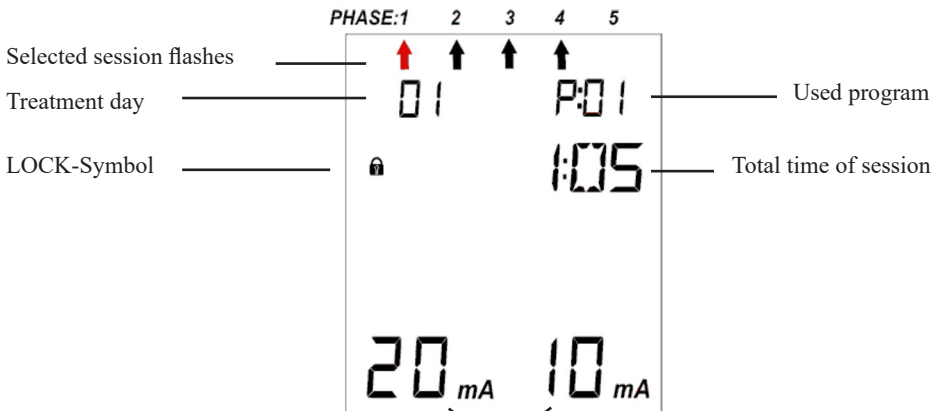To get more detailed information press the + button of the left channel (A). You will see the data of the first session of the first day.


Average used intensity per channel

## Reading the Statistics

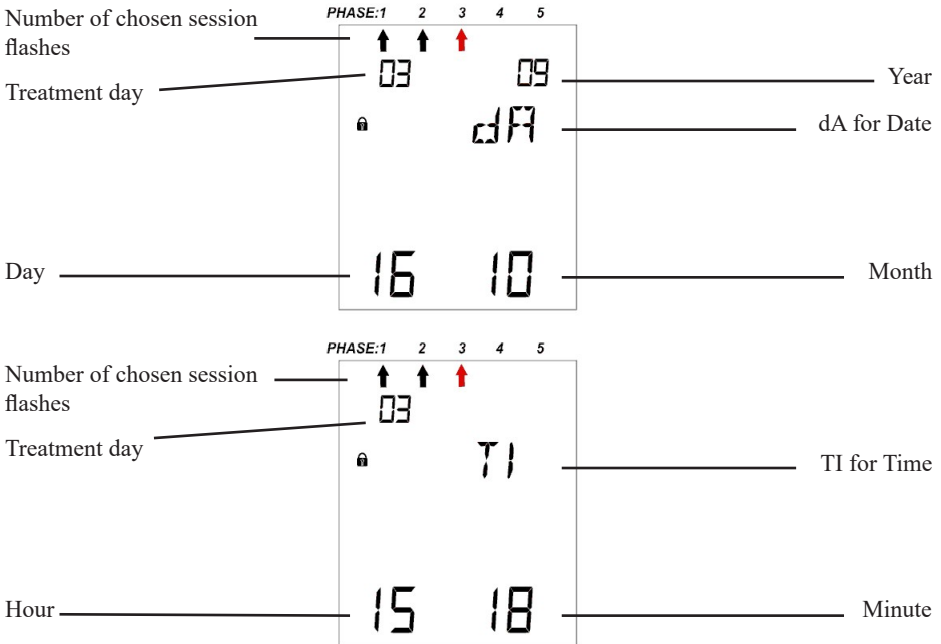The next sessions of the selected day are displayed - one after the other - by pressing the „SET“-button. Each next day can be obtained by pressing the + button of channel A again. If this button is held for at least 3 seconds, the device returns to global statistics.

If you require information about time and date of one single session please press any of the right +/--buttons (channel B). With these buttons you can scroll between session data, date and time.

The diagrams illustrate the display layout for session statistics and time/date information. The top diagram shows the date (03.09) and time (16:10) with session flashes (03) and a lock icon. The bottom diagram shows the time (15:18) and session flashes (03) with a lock icon. Labels indicate the meaning of each part of the display.

**Top Diagram (Date and Time):**

- PHASE:1 2 3 4 5 (with arrows pointing to 1, 2, 3)
- Number of chosen session flashes: 03
- Treatment day: 03
- Year: 09
- dA for Date: 09
- Day: 16
- Month: 10

**Bottom Diagram (Time):**

- PHASE:1 2 3 4 5 (with arrows pointing to 1, 2, 3)
- Number of chosen session flashes: 03
- Treatment day: 03
- TI for Time: 11
- Hour: 15
- Minute: 18

The data stored in memory is retained after viewing, so that therapy-control is possible anytime. A maximum of 60 days of data can be stored with 5 sessions per day. When 60 days of data are reached, consecutive days will overwrite the earliest days. The day number will continue increasing (61, 62, ...) and will be displayed when the unit is unlocked until the **BioBravo** is reset.

If you press the PRG button for three seconds, you return to the home screen of the selected program. The device will not be locked. If the therapy shall continue with only one program available, the LOCK-function must be reactivated after reading (s. p. 53).

The memory should be emptied completely before a new person is using the device. After pressing the UNLOCK-button you have to press SET twice for 5 seconds each. All individual settings in programs 10-12 will be reset as well. The **BioBravo** is set to it's default state and all statistics are deleted.

## Trouble Shooting Guide

<b>Fault</b>	<b>Possible Cause</b>	<b>Solution</b>
Trip off - Electrode symbol flashing, current won't increase above 8 mA)	Electrode lost adhesion to skin	Check electrodes and replace if necessary.
	Electrode too dry and loses adhesion to skin	Replace electrode.
	Lead wire not connected properly.	Check lead wire connections and ensure they are fully inserted.
	Lead wire is faulty.	Replace lead wire.
	Device is damaged.	Return device to manufacturer for repair.
Device does not power on.	Flat battery or no batteries installed in device.	Check / Replace batteries.
	Battery installed in wrong orientation.	Remove batteries and install according to diagrams in battery compartment.
	Battery connector damaged or flattened.	Remove batteries, check metal parts in battery compartment for damage.
	Device is damaged.	Return to manufacturer for repair.
Device powers off during use.	Batteries almost flat.	Check battery icon during use and replace batteries if low.
	Device is damaged.	Return to manufacturer for repair.
Current output is unstable	Device is in use near a device emitting strong radio or magnetic interference.	Move at least four meters away from the offending device and resume treatment.
Treatment is painful or uncomfortable when using skin electrodes.	Patients skin has cuts or abrasions in the area of electrode placement.	Discontinue use until skin is fully healed.
Treatment is painful or uncomfortable when using a probe.	Patient reaction to probe materials.	Discontinue use and consult a doctor for an allergy check if discomfort continues.
Skin becomes red after treatment.	Device is being used for periods which are too long for the patient.	Use device for a shorter treatment duration.
Treatment is uncomfortable, tingling sensations during use.	Intensity too high for required treatment.	Reduce intensity setting until treatment is comfortable. Note: Tingling sensations are normal and should be felt.

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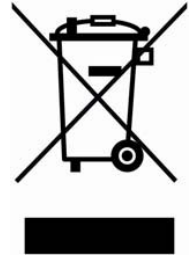
In accordance with European regulations [1], used electrical and electronic equipment may not be discarded in household or unsorted municipal waste. Use separate collection facilities. Contact your local government or council for information regarding the collection systems available. If electrical appliances are disposed of in landfills or dumps, hazardous substances can leak into the ground water and get into the food chain damaging the health and well being of people and animals.

[1] European Community directive 2012/19/EU on waste electrical and electronic equipment.

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The batteries should be replaced when they are drained or stop functioning. The batteries supplied with the **BioBravo** are recyclable. Remove the batteries from the unit and follow your local recycling guidelines.

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A list of literature regarding the topics of this manual can be found on our website „[www.mtrplus.com](http://www.mtrplus.com)“.

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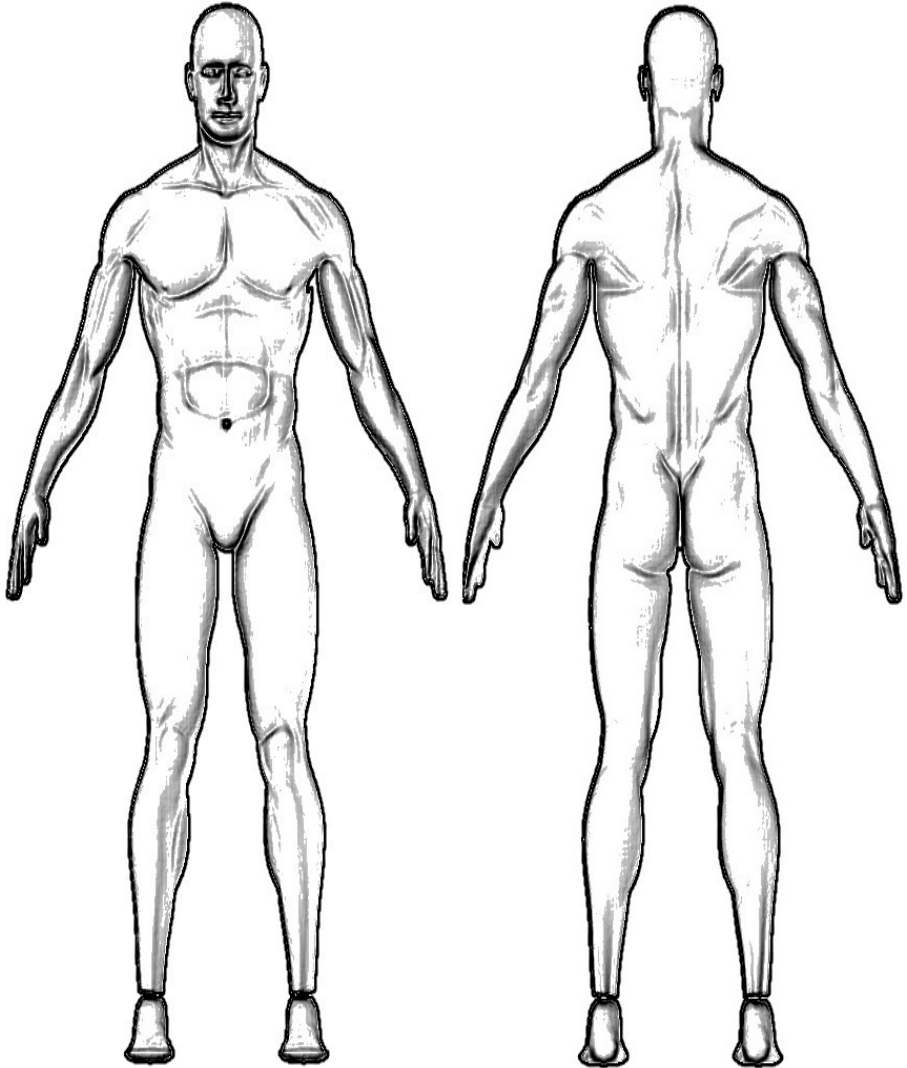
Reserved for changes.

Printing or copy of this manual - also in an abridged version - is forbidden and can only be allowed with a written approval of “MTR+ Vertriebs GmbH”.

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## Electromagnetic Compatibility

If the device is used according to the instructions given in this manual, it corresponds to the safety requirements for medical electrical systems IEC 60601-1. **BioBravo** was checked and is in accordance with the limits set for medical products in the standard IEC 60601-1-2:2014. These limits serve to the safety of the user regarding electromagnetic disturbances.

The **BioBravo** is a Class B device and is suitable for home healthcare environments as well as typical clinical environments.

Guidance and manufacturer's declaration - electromagnetic emissions		
The <b>BioBravo</b> unit is intended for use in the electromagnetic environment specified below. The customer or the user of the <b>BioBravo</b> should ensure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The <b>BioBravo</b> uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The <b>BioBravo</b> is suitable for use in all establishments, including domestic and medical establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not Applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not Applicable	

Please adhere to these warnings when using the device:

Don't use **BioBravo** near active HF surgical equipment or RF shielded rooms used for magnetic resonance imaging, where the intensity of EM disturbances is high.

Use of this equipment adjacent to, or placement on top of other equipment should be avoided in case the output of the **BioBravo** is effected. If this is necessary, Both the **BioBravo** and the other equipment should be monitored to verify that they are both operating normally.

Use of accessories and lead wires other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used not closer than 30 cm (12 inches) to any part of the **BioBravo**, including lead wires specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

The performance of the **BioBravo** can be effected if exposed to frequencies of 85 Mhz to 340 Mhz. If the **BioBravo** is exposed to RF devices operating at these frequencies, the output of the current may be reduced. If this occurs, move to a position further away from the RF equipment, the output will stabilize when the **BioBravo** is moved to a suitable position.

## Electromagnetic Compatibility

<b>Guidance and manufacturer's declaration - electromagnetic Immunity</b>			
The <b>BioBravo</b> is intended for use in the electromagnetic environment specified below. The customer or the user of the <b>BioBravo</b> should assure that it is used in such an environment			
Immunity Test	IEC 60601 Test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact  ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact  ±2 kV, ±4 kV, ±8 kV, ±15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %
Electrical fast transient/burst IEC 61000-4-4	Power supply lines: ±2 kV input/output lines: ±1 kV	Not Applicable	<b>BioBravo</b> is powered by batteries
Surge IEC 61000-4-5	line(s) to line(s): ±1 kV. line(s) to earth: ±2 kV. 100 kHz repetition frequency	Not Applicable	<b>BioBravo</b> is powered by batteries
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° phases 0% U <sub>T</sub> for 0.5 cycle  And at single phase 0° 0% U <sub>T</sub> for 1 cycle 70% U <sub>T</sub> for 25/30 cycles 0% U <sub>T</sub> for 250 cycles 0% U <sub>T</sub> for 300 cycles	Not Applicable	<b>BioBravo</b> is powered by batteries
Power frequency (50/60Hz) magnetic field IEC 61000-4-8	30 A/m 50Hz/60Hz	30 A/m 50Hz/60Hz	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Conducted RF IEC61000-4-6	150 KHz to 80 MHz: 3 Vrms 6 Vrms (in ISM and amateur radio bands) 80 % AM at 1 kHz	150 KHz to 80 MHz: 3 Vrms 6 Vrms (in ISM and amateur radio bands) 80 % AM at 1 kHz	Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF IEC61000-4-3	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	
NOTE 1: U <sub>T</sub> is the a.c. mains voltage prior to application of the test level. NOTE 2: At 80 MHz the higher frequency range applies. NOTE 3: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

## Electromagnetic Compatibility

### Guidance and manufacturer's declaration - electromagnetic Immunity

The **BioBravo** is intended for use in the electromagnetic environment specified below. The customer or the user of the **BioBravo** should assure that it is used in such an environment

	Test Frequency (MHz)	Band (MHz)	Service	Modulation	Modulation (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
Radiated RF IEC61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	385	380 – 390	TETRA 400	Pulse modulation 18 Hz	1,8	0,3	27
	450	380 – 390	GMRS 460, FRS 460	FM c) ± 5 kHz deviation 1 kHz sine	2	0,3	28
	710	704 – 787	LTE Band 13, 17	Pulse modulation 217 Hz	0,2	0,3	9
	745						
	780						
	810	800 – 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	2	0,3	28
	870						
	930						
	1720	1 700 – 1 990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation 217 Hz	2	0,3	28
	1845						
	1970						
	2450	2 400 – 2 570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0,3	28
	5240	5 100 – 5 800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0,2	0,3	9
	5240						
	5785						

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